

# **BILIRUBIN (T&D) TEST KIT (SR)**

**INTENDED USE:** Reagent kit for the quantitative determination of Total and Direct Bilirubin in serum.

CLINICAL SIGNIFICANCE: Approximately 80 - 85% of the Bilirubin produced is derived from the heme moiety of the haemoglobin released from aging erythrocytes in the reticuloendothelial cells. Bilirubin bound to albumin is transported into the liver where it is rapidly conjugated with glucuronide to increase its solubility. Then it is excreted into biliary canaliculi and hydrolyzed in the gastrointestinal tract. Unconjugated Bilirubin serum concentration increases in case of over production of Bilirubin (acute or chronic haemolytic anemias) and in case of disorders of Bilirubin metabolism and transport defects (impaired uptake by liver cells: Gilbert's syndrome; defects in the conjugation reaction: Crigler-Najjar syndrome). Reduced excreation (hepatocellular damage: hepatitis, cirrhosis....; Dubin Jhonson and Rotor syndrome) and obstruction to the flow of bile (most often produced by gallstones or by tumours) induce an important elevation of conjugated Bilirubin and in a minor extent an increase of unconjugated Bilirubin (conjugated hyperbilirubinemia).

# **REAGENT COMPOSITION:**

Reagent T 1 : Total Bilirubin Reagent. Reagent D 1 : Direct Bilirubin Reagent.

## **MATERIALS REQUIRED BUT NOT PROVIDED:**

- · Clean & Dry Glassware.
- · Micropipettes & Tips.
- · Colorimeter or Bio-Chemistry Analyzer.

**SAMPLES:** Serum free of hemolysis. Heparinized plasma. Care must be taken to fill heparinized tubes according to the manufacturer's instructions. An insufficient filling may lead to erroneous results Protect the samples from light before & during the analysis.

# **STABILITY OF REAGENT:**

When Stored tightly closed at 2° To 8°C protected from light and contaminations prevanted during their use; reagents are stable up to the expiry date stated on the lable.

WORKING REAGENT: This Reagent is ready for use.

# **ASSAY PROCEDURE:**

Total Bilirubin:

	Blank	Sample
Direct Bilirubin D1	1000 μΙ	1000 μΙ
Sample	-	50 µl

# Direct Bilirubin:

	Blank	Sample
Total Bilirubin T1	1000 μΙ	1000 μl
Sample	-	50 μl

Mix and read the absorbance of the tests against their respective reagent blanks after a 5 min. incubation at 37°C.

### **GENERAL SYSTEM PARAMETERS:**

End Point	
546 nm	
630 nm	
1cm	
37°C	
Reagent Blank	
1000 μΙ	
50 μΙ	
5 Minutes	
30	
15	
20 mg/dl	

#### **CALCULATION:**

Total Bilirubin (mg/dl) = (Abs. Test - Abs. Blank) X 30 Direct Bilirubin (mg/dl) = (Abs. Test - Abs. Blank) X 15

LINEARITY: Reagent is Linear up to 20 mg/dl.

Dilute the sample appropriately and re-assay if Total or Direct Bilirubin concentration exceeds 20 mg/dl.

Multiply result with dilution factor.

# **REFERENCE NORMAL VALUE:**

Total Bilirubin: (Adults and Children over 10 days)

0.3 - 1.2 mg/dl

Direct Bilirubin : < 0.2 mg/dl

**QUALITY CONTROL:** For accuracy it is necessary to run known controls with every assay.

**SENSITIVITY / LIMIT DETECTION :** The Lower Limit of detection is 0.05 mg/dl.

# **BIBLIOGRAPHY:**

Tietz, N. W., Clinical guide to laboratory tests. 3rd Ed., (W. B. Saunders eds. Philadelphia USA), (1995), 90. Vassault, A., et al., Protocole de validation de techniques. (Document B, stade 3), Ann. Biol. Clin., (1986), 44, 686.



